

Quick Start Guide. SIMPLE. EASY-TO-USE.



- 1 Wash hands with warm, soapy water. Dry hands thoroughly.



- 2 Remove test strip from vial and close the vial immediately. Insert test strip into the test port. The meter turns on.



- 3 Lance your finger. With test strip still in meter, touch tip of test strip to blood drop. Allow blood to be drawn into the test strip. Remove test strip from drop when count down begins.




- 4 Glucose result will be displayed. Remove test strip and meter will turn off. Disposable - Discard entire unit after final test.

For performing a Control Solution Test:

Follow instructions on the Control Solution Instructions for Use.

When testing with Control Solution (A), compare the Meter result (C) to the control range (Level 0 or 1) located on the test strip vial label (B).

These are NOT ranges for blood glucose results.

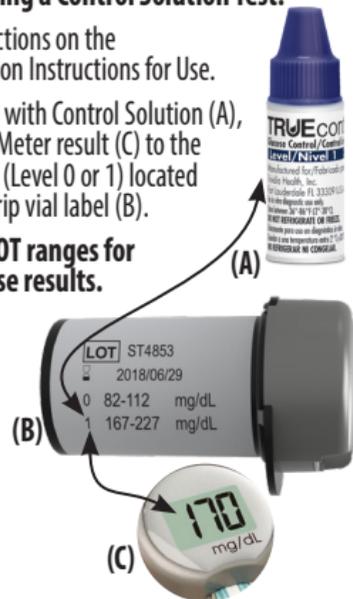

WARNING! Do not use Test Strips or Meter for blood glucose testing if Control results are not within Control range.

Sidekick® All-In-One Blood Glucose Testing System is to be used ONLY with Sidekick® All-In-One Blood Glucose Testing System Test Strips and TRUEcontrol® Control Solution. • For questions on using this product, please call 1-800-803-6025, Monday - Friday, 8AM-8PM EST. For additional information on using the Sidekick® All-In-One Blood Glucose System please refer to the Instructions for Use and the Control Solution Instructions for Use.

WARNING: Upon opening the carton, examine the product for missing, damaged or broken parts. Ensure the vial cap is securely closed. If the product is damaged or the vial cap is not closed, DO NOT use the test strips for testing; product may give inaccurate results. Contact Trividia Health Customer Care at 1-800-803-6025 for replacement and assistance.