
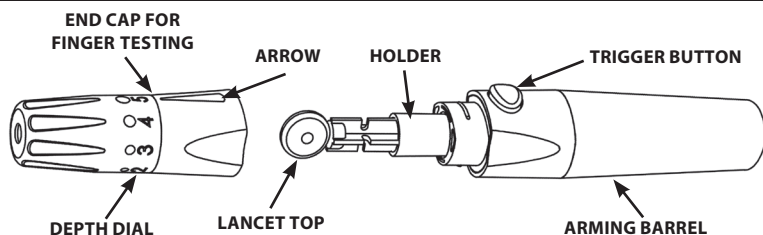
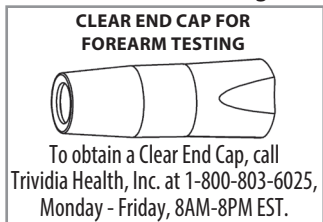
# LANCING DEVICE INSTRUCTIONS FOR USE


For obtaining capillary whole blood for testing.

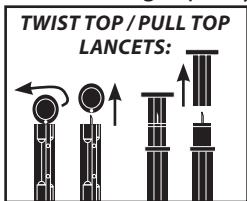- WARNING!** The Lancing Device is for single patient use **ONLY**. **DO NOT** share your Lancing Device with others, including family members. **DO NOT** use on multiple patients. **ALL** parts of the Lancing Device are considered biohazardous and can potentially transmit infectious diseases, even after you have performed cleaning and disinfection.
- CAUTION!** **Do not use lancet if lancet top has been previously removed.** We recommend that you wash your hands thoroughly with soap and water after handling the Meter, Lancing Device, or Test Strips to prevent the transfer of blood-borne viruses or infectious agents.

## - R E A D Y -

- Select area to be lanced. Wash with soap and warm water, rinse and dry thoroughly.
- Pull off End Cap.  
Push Lancet firmly into Holder.
- Remove Lancet Top.  
**Note: Save Top for safe disposal of used Lancet.**
- Replace End Cap.  
**Note: Use Clear End Cap for lancing forearm.**
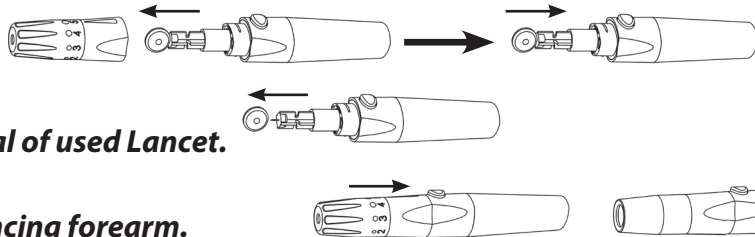


To obtain a Clear End Cap, call Trividia Health, Inc. at 1-800-803-6025, Monday - Friday, 8AM-8PM EST.

## - S E T -



### For Finger Testing:

- Turn Depth Dial to desired lancing depth.  
**Note: Finer adjustments can be made by setting Arrow between numbers.**  
Lancing Depth Guide:  
• 1-2 for soft / thin skin  
• 3-4 for average skin  
• 4-5 for thick / calloused skin
- Lower hand to waist level. Gently massage finger to get blood flowing.
- Hold End Cap in one hand and Arming Barrel in other. Gently pull Arming Barrel until a click is heard.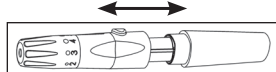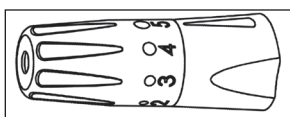


### For Forearm Testing:

- Use Clear End Cap in place of regular Depth Dial End Cap, but if Clear End Cap is not available, select #5 on Depth Dial.
- Rub area vigorously or apply warm compress to stimulate blood flow.
- Hold End Cap in one hand and Arming Barrel in other. Gently pull Arming Barrel until a click is heard.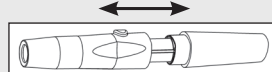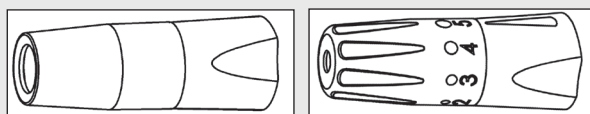


## - L A N C E -

- Place Tip firmly against finger on other hand.  
**Note: For best results, lance in shaded areas.**
- Press Trigger Button.
- Gently massage finger from palm to fingertip to form blood drop.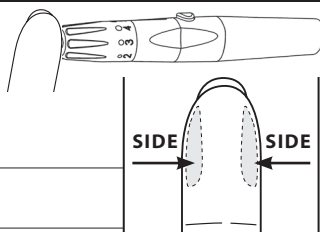

- Place Tip firmly against forearm.  
**Note: For best results, lance in shaded area.**
- Press Trigger Button.
- Apply a firm pressure on the Lancing Device until an adequate blood drop is formed.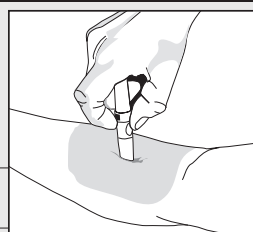

### Removing Used Lancet:


- Pull off End Cap. 2. Using saved Top, recap Lancet. 3. Remove from Holder and dispose of Lancet in appropriate container.

### Lancing Device Cleaning and Disinfecting

Cleaning and disinfecting the lancing device is a two step process. The first step is **Cleaning**. Cleaning removes any blood, liquids, dust or dirt from the lancing device. **Cleaning must be done before Disinfecting.** The second step is **Disinfecting**. Disinfecting removes most, but not all possible infectious agents (bacteria or viruses) from the lancing device, including blood-borne pathogens.

**Caution!** Clean and disinfect the lancing device at least once a week. Lancing device may be cleaned and disinfected once a week for up to 3 years. If blood gets on the lancing device, clean and disinfect it before anyone else touches it. If someone is helping you test your blood or performs a test on you, do the cleaning and disinfecting before allowing anyone else to handle it.

- Wash hands thoroughly with soap and water or wear disposable gloves.
- To Clean:** Using **ONLY** Super Sani-Cloth Wipes (EPA\* reg. no. 9480-4), thoroughly wipe the entire outside surface of the lancing device with one wipe. Discard the used wipe.
- Remove a fresh Super Sani-Cloth Wipe and thoroughly wipe the entire outside surface of the lancing device by wiping in circles 3 times. Make sure that you go over the entire lancing device.
- To Disinfect:** Using fresh wipes, make sure that all outside surfaces of the lancing device stay wet by patting with the wipe for 2 minutes. Discard used wipes. Let lancing device air dry completely before using.
- Gently pull back Arming Barrel and press the Trigger Button. A click is heard if the lancing device is working properly.
- Wash hands thoroughly again.

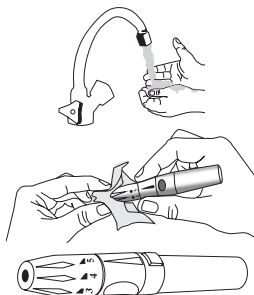**Caution!** Other disinfectants have not been tested. The effect of other disinfectants used interchangeably has not been tested with the lancing device. Use of disinfectants other than Super Sani-Cloth Wipes may damage lancing device.

Do not use lancing device and call Customer Care for assistance at 1-800-803-6025, available Monday - Friday, 8AM - 8PM EST, if:

- ~ Markings on lancing device are coming off or missing,
- ~ Trigger Button is hard to push or does not work,
- ~ Arming Barrel does not click when gently pulled back.

Only use Super Sani-Cloth Wipes to clean and disinfect the lancing device. Super Sani-Cloth Wipes may be purchased at the following places: • Amazon.com • Staples.com • Walmart.com

Super Sani-Cloth Wipes have been tested on the lancing device for a total of 1095 cleaning and disinfecting cycles, which is equal to cleaning and disinfecting the lancing device once a day for a 3 year period. Lancing device may be cleaned and disinfected up to 7 cycles per week for 3 years.

\*Environmental Protection Agency

### Important Notes Regarding Forearm Samples Used in Testing Blood Glucose:\*

- Check with your Doctor or Diabetes Healthcare Professional to see if forearm testing is right for you.
- Blood glucose results from forearm are not always the same as results from fingertip. Do not use forearm for testing when blood sugar is rapidly rising or falling, e.g., within 2 hours of eating, after taking insulin, immediately after exercise, or when you are ill or under stress. Use fingertip sample for testing.
- Do not use forearm samples for testing if you think your blood sugar is low, if you don't regularly experience symptoms when your blood glucose is low, or if your results from forearm samples do not match the way you feel. Use fingertip sample for testing.

**Note: Lancing Device and Clear End Cap are for self-testing and intended for use by one patient only. Not suitable for use by healthcare or care workers.**

- Refer to Owner's Booklet for testing instructions. •

Manufactured for:

**TRIVIDIA**  
HEALTH™  
Fort Lauderdale, FL 33309 USA  
www.trividiahealth.com  
1-800-803-6025 • 1-954-677-4599

1. U.S. Food and Drug Administration. Blood Glucose Meters, Getting the Most Out of Your Meter. [Electronic Version]. Retrieved December 22, 2009 from <http://www.fda.gov/MedicalDevices/Safety/AlertsandNotices/TipsandArticlesonDeviceSafety/ucm109371.htm>

© 08/2018 Trividia Health, Inc. The Trividia Health logo is a trademark of Trividia Health, Inc. Made in China • U.S. and International Patents Pending. • M2TVH06 Rev. 42

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